THUASNE®

Ligastrap® Malleo

fr	Chevillère élastique proprioceptive avec sangles stabilisatrices	5
en	Proprioceptive elastic ankle brace with stabilising straps	6
de	Elastische propriozeptive Knöchelbandage mit Stabilisierungsbändern	7
nl	Proprioceptieve elastische enkelbrace met stabiliserende banden	9
it	Cavigliera elastica proprioceppiva con cinghie stabilizzatrici	10
es	Tobillera elástica propioceptiva con cinchas estabilizadoras	12
pt	Tornozoleira elástica propioceptiva com fitas estabilizadoras	13
da	Proprioceptisk ankelbind med stabiliserende stropper	14
fi	Proprioseptiivinen joustava nilkkatuki, jossa stabiloivat hihnat	16
sv	Proprioceptiv elastisk ankelortos med stabiliserande remmar	17
el	Ελαστική ιδιοδεκτική επιστραγαλίδα με μάντες σταθεροποίησης	19
cs	Elastická proprioceptivní kotníková ortéza se stabilizačními pásky	20
pl	Elastyczna proprioceptywna orteza skokowo-stopowa z taśmami stabilizującymi	21
lv	Proprioceptīvs elastīgs potītes pārsējs ar stabilizēšanas siksnām	23
lt	Tamprus propriocepriją gerinantis čiurnos įtvaras su stabilizuojančiais diržais	24
et	Proprioseptiivne elastne hüppeliigesetugi stabiliseerivate rihmadega ..	26
sl	Elastična proprioceptivna opornica s trakovi za stabiliziranje	27
sk	Proprioceptívna elastická členková ortéza so stabilizačnými páskami	28
hu	Proprioceptív rugalmas bokarögzítő stabilizáló pántokkal	30
bg	Проприоцептивна еластична шина за глезен със стабилизиращи каишки	31
ro	Glezneră elastică proprioceptivă cu fașe stabilizatoare	33
ru	Эластичный проприоцептивный ортез на лодыжку со стабилизирующими ремнями	34
hr	Elastični proprioceptivni steznik za gležanj s remenima za stabilizaciju ..	36
zh	帶穩定搭扣的彈力本体感護踝	37
ar	دعامة كاحل مرنة متعلقة باستقبال الحس العميق مع أحزمة للتثبيت	39

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

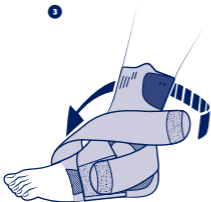
1


2

3

4

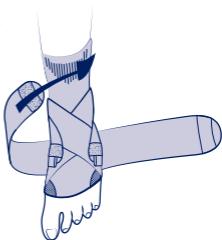5

	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
ru	Окружность голени
hr	Opseg gležnja
zh	腳踝尺寸
ar	محيط الكاحل

fr	Maintien	Compression	Effet proprioceptif
en	Support	Compression	Proprioceptive effect
de	Halt	Kompression	Propriozeptive Wirkung
nl	Fixatie	Compressie	Proprioceptieve werking
it	Sostegno	Compressione	Effetto propriocettivo
es	Sujeción	Compresión	Efecto propioceptivo
pt	Manutenção	Compressão	Efeito proprioceptivo
da	Støtte	Kompression	Proprioceptisk effekt
fi	Tuki	Kompressio	Liikeaistiin kohdistuva vaikutus
sv	Stöd	Kompression	Proprioceptiv effekt
el	Συγκράτηση	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας
cs	Stabilizace	Komprese	Proprioceptivní účinek
pl	Podtrzymywanie	Kompresja	Propriocepcja
lv	Atbalsts	Kompresija	Proprioceptīvs efekts
lt	Palaikymas	Kompresija	Propriocepčinis poveikis
et	Hooldus	Kompressioon	Proprioceptiivne toime
sl	Oprijem	Kompresija	Proprioceptivni učinek
sk	Podpora	Kompresia	Proprioceptívny účinok
hu	Támasz	Kompresszió	Proprioceptív hatás
bg	Поддръжка	Компресия	Проприоцептивен ефект
ro	Susținere	Compresie	Efect proprioceptiv
ru	Стабилизация	Компрессия	Проприоцептивный эффект
hr	Potpورا	Kompresija	Učinak proprioceptivnog vježbanja
zh	稳定	压紧	本体感
ar	الدعم	الضغط	مفعول استقبال الحس العميق

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Conserver cette notice.

en

PROPRIOCEPTIVE ELASTIC ANKLE BRACE WITH STABILISING STRAPS

Description/Destination

This device is a proprioceptive elastic ankle brace providing joint support. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

This device can be used as a proprioceptive support for rehabilitation and particularly when resuming sports activities.

Composition

Polyamide - elastane - polyester - elastodiène - polyuréthane.

Properties/Mode of action

Elastic compressive knitting (class 3 - 20-36 mmHg / 27,1-48 hPa).

Double lateral support strapping to maintain flexion-extension.

Removable lateral support strap.

Tabs to make it easier to pull on.

Indications

Mild ankle sprain (acute phase).

Postoperative, post-traumatic residual pain and/or edema.

Joint instability/laxity.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient. Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Slip on the ankle brace. The top comprises the closure tab.

Close this tab (with the self-adhesive strip) to obtain a snug fit.

Pull the first strap, wrap it around the ankle ❶ and fix it on the felt pad located at the base ❷.

Repeat this operation with the second strap ❸: the ankle must be firmly supported laterally with unhindered flexion-extension.

If necessary, the ankle support may be reinforced. To do this, lay out the removable strap on the ground, with the self-adhesive parts upwards. Then place the foot on its midpart ❹, and pull the two ends up.

Make an "8" crossover above the foot and fix the ends of the strap on the felt pad at the top of the ankle brace ❺.

For comfort during wear make sure the straps are not creased.

Care

The product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Machine washable at 40°C (delicate programme). The device has been tested over 20 wash cycles. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2023402 (2025-01)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186