**THUASNE®**

# MalleoSoft

<b>fr</b>	<b>Chevillère élastique proprioceptive</b> .....	5
<b>en</b>	<b>Proprioceptive elastic ankle brace</b> .....	6
<b>de</b>	<b>Elastische propriozeptive Knöchelbandage</b> .....	7
<b>nl</b>	<b>Proprioceptieve elastische enkelbrace</b> .....	9
<b>it</b>	<b>Cavigliera elastica proprioceittiva</b> .....	10
<b>es</b>	<b>Tobillera elástica propioceptiva</b> .....	12
<b>pt</b>	<b>Tornozeleira elástica proprioceptiva</b> .....	13
<b>da</b>	<b>Proprioceptisk, elastisk ankelbind</b> .....	15
<b>fi</b>	<b>Proprioseptiivinen joustava nilkkatuki</b> .....	16
<b>sv</b>	<b>Proprioceptiv elastisk ankelortos</b> .....	18
<b>el</b>	<b>Ελαστική ιδιοδεκτική επιστραγαλίδα</b> .....	19
<b>cs</b>	<b>Elastická proprioceptivní kotníková ortéza</b> .....	20
<b>pl</b>	<b>Elastyczna proprioceptywna orteza skokowo-stopowa</b> .....	22
<b>lv</b>	<b>Proprioceptīvs elastīgs potītes pārsējs</b> .....	23
<b>lt</b>	<b>Tamprus propriocepriją gerinantis čiurnos įtvaras</b> .....	25
<b>et</b>	<b>Proprioseptiivne elastne hüppeliigesetugi</b> .....	26
<b>sl</b>	<b>Elastična proprioceptivna opornica</b> .....	27
<b>sk</b>	<b>Elastická proprioceptívna členková ortéza</b> .....	29
<b>hu</b>	<b>Proprioceptív rugalmas bokarögzítő</b> .....	30
<b>bg</b>	<b>Проприоцентивна еластична шина за глезен</b> .....	32
<b>ro</b>	<b>Glezniere elastică proprioceptivă</b> .....	33
<b>ru</b>	<b>Эластичный проприоцентивный ортез на лодыжку</b> .....	34
<b>hr</b>	<b>Elastični proprioceptivni steznik za gležanj</b> .....	36
<b>zh</b>	<b>彈力本体感护踝</b> .....	37
<b>ar</b>	<b>دعامة كاحل مرنة لاستقبال الحس</b> .....	39

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

<b>fr</b>	Tour de cheville
<b>en</b>	Ankle circumference
<b>de</b>	Knöchelumfang
<b>nl</b>	Omvang van de enkel
<b>it</b>	Giro caviglia
<b>es</b>	Circunferencia del tobillo
<b>pt</b>	Perímetro do tornozelo
<b>da</b>	Anklens omkreds
<b>fi</b>	Nilkanympärys
<b>sv</b>	Runt fotleden
<b>el</b>	Περίμετρος αστραγάλου
<b>cs</b>	Obvod kotníku
<b>pl</b>	Obwód kostki
<b>lv</b>	Potītes apkārtmērs
<b>lt</b>	Kulkšnies apimtis
<b>et</b>	Pahkluu ümbermõõt
<b>sl</b>	Obseg gležnja
<b>sk</b>	Obvod členka
<b>hu</b>	Boka kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința gleznei
<b>ru</b>	Окружность голени
<b>hr</b>	Opseg gležnja
<b>zh</b>	腳踝尺寸
<b>ar</b>	محيط الكاحل

<b>fr</b>	Maintien	Compression	Effet proprioceptif
<b>en</b>	Support	Compression	Proprioceptive effect
<b>de</b>	Halt	Kompression	Propriozeptive Wirkung
<b>nl</b>	Fixatie	Compressie	Proprioceptieve werking
<b>it</b>	Sostegno	Compressione	Effetto propriocettivo
<b>es</b>	Sujeción	Compresión	Efecto propioceptivo
<b>pt</b>	Manutenção	Compressão	Efeito proprioceptivo
<b>da</b>	Støtte	Kompression	Proprioceptisk effekt
<b>fi</b>	Tuki	Kompressio	Liikeaistiin kohdistuva vaikutus
<b>sv</b>	Stöd	Kompression	Proprioceptiv effekt
<b>el</b>	Συγκράτηση	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας
<b>cs</b>	Stabilizace	Komprese	Proprioceptivní účinek
<b>pl</b>	Podtrzymywanie	Kompresja	Propriocepcja
<b>lv</b>	Atbalsts	Kompresija	Proprioceptīvs efekts
<b>lt</b>	Palaikymas	Kompresija	Propriocepčinis poveikis
<b>et</b>	Hoodus	Kompressioon	Propriotseptiivne toime
<b>sl</b>	Oprajem	Kompresija	Proprioceptivni učinek
<b>sk</b>	Podpora	Kompresia	Proprioceptívny účinok
<b>hu</b>	Támasz	Kompresszió	Proprioceptív hatás
<b>bg</b>	Поддръжка	Компресия	Проприоцептивен ефект
<b>ro</b>	Susținere	Compresie	Efect proprioceptiv
<b>ru</b>	Стабилизация	Компрессия	Проприоцептивный эффект
<b>hr</b>	Potpora	Kompresija	Učinak proprioceptivnog vježbanja
<b>zh</b>	稳定	压紧	本体感
<b>ar</b>	الدعم	الضغط	مفعول استقبال الحس العميق

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---

en

## **PROPRIOCEPTIVE ELASTIC ANKLE BRACE**

### **Description/Destination**

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

This device can be used as a proprioceptive support for rehabilitation and particularly when resuming sports activities.

### **Composition**

Polyamide - cotton - elastane.

### **Properties/Mode of action**

Elastic compressive knitting (class 2 - 15-20 mmHg / 20,1-27 hPa).

Comfort zone over the ankle bones.

### **Indications**

Postoperative, post-traumatic residual pain and/or edema.

### **Contraindications**

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

## **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

## **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## **Instructions for use/Application**

Pull the ankle brace over the ankle joint, like a sock.

Place the malleoli correctly, on the comfort zones.

## **Care**

The product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Machine washable at 40°C (delicate programme).

The device has been tested over 20 wash cycles. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat.

Dry away from any direct heat source (radiator, sun, etc.).

## **Storage**

Store at room temperature, preferably in the original packaging.

## **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2023902 (2025-01)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186