

**THUASNE®**

# Silistab® Achillo

<b>fr</b>	Chevillère élastique proprioceptive pour tendon d'Achille .....	5
<b>en</b>	Proprioceptive elastic ankle brace for Achilles tendon.....	6
<b>de</b>	Elastische propriozeptive Knöchelbandage für Achillessehne .....	7
<b>nl</b>	Proprioceptieve elastische enkelbrace voor achillespees.....	9
<b>it</b>	Cavigliera elastica proprioceittiva per tendine di Achille.....	10
<b>es</b>	Tobillera elástica propioceptiva para tendón de Aquiles.....	12
<b>pt</b>	Tornozeleira elástica proprioceptiva para tendão de Aquiles .....	13
<b>da</b>	Proprioceptisk, elastisk ankelbind til akillessene .....	15
<b>fi</b>	Proprioseptiivinen joustava nilkkatuki akillesjanteelle.....	16
<b>sv</b>	Proprioceptiv elastisk ankelortos för hälsenan.....	17
<b>el</b>	Ελαστική ιδιοδεκτική επιστραγαλίδα για αχίλλειο τένοντα.....	19
<b>cs</b>	Elastická proprioceptivní kotníková ortéza na Achillovu šlachu .....	20
<b>pl</b>	Elastyczna proprioceptywna orteza skokowo-stopowa z wkładkami odciążającymi .....	21
<b>lv</b>	Proprioceptīvs elastīgs potītes pārsējs Ahileja cīpslai .....	23
<b>lt</b>	Tamprus propriocepriją gerinantis čiurnos įtvaras Achilo sausgyslei.....	24
<b>et</b>	Proprioseptiivne elastne hüppeliigesetugi Achilleuse kõõluse jaoks.....	26
<b>sl</b>	Elastična proprioceptivna opornica za Ahilovo tetivo .....	27
<b>sk</b>	Elastická proprioceptívna členková ortéza na Achillovu šľachu.....	28
<b>hu</b>	Proprioceptív rugalmas bokarögzítő az Achilles-ínhez .....	30
<b>bg</b>	Проприоцептивна еластична шина за глезен за ахилесовото сухожилие .....	31
<b>ro</b>	Glezneră elastică proprioceptivă de protecție a tendonului lui Ahile.....	33
<b>ru</b>	Эластичный проприоцептивный ортез для разгрузки ахиллова сухожилия .....	34
<b>hr</b>	Elastični proprioceptivni steznik za gležanj za Ahilovu tetivu.....	36
<b>zh</b>	跟腱弹力本体感护踝 .....	37
<b>ar</b>	دعامة كاحل مرنة متعلقة باستقبال الحس العميق لوتر العرقوب.....	39

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung -  
Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de  
utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning -  
Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas  
instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo -  
Pokyny na používanie - Használati utasítás - Инструкции за употреба -  
Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu -  
使用指南 - إرشادات الاستعمال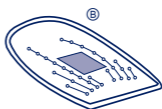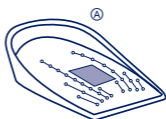

	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

<b>fr</b>	Tour de cheville
<b>en</b>	Ankle circumference
<b>de</b>	Knöchelumfang
<b>nl</b>	Omvang van de enkel
<b>it</b>	Giro caviglia
<b>es</b>	Circunferencia del tobillo
<b>pt</b>	Perímetro do tornozelo
<b>da</b>	Anklens omkreds
<b>fi</b>	Nilkanympärys
<b>sv</b>	Runt fotleden
<b>el</b>	Περίμετρος αστραγάλου
<b>cs</b>	Obvod kotníku
<b>pl</b>	Obwód kostki
<b>lv</b>	Potītes apkārtmērs
<b>lt</b>	Kulkšnies apimtis
<b>et</b>	Pahkluu ümbermõõt
<b>sl</b>	Obseg gležnja
<b>sk</b>	Obvod členka
<b>hu</b>	Voka kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința gleznei
<b>ru</b>	Окружность голени
<b>hr</b>	Opseg gležnja
<b>zh</b>	脚踝尺寸
<b>ar</b>	محيط الكاحل

<b>fr</b>	Compression	Effet proprioceptif	Décharge
<b>en</b>	Compression	Proprioceptive effect	Off-loading
<b>de</b>	Kompression	Propriozeptive Wirkung	Entlastung
<b>nl</b>	Compressie	Proprioceptieve werking	Losmaken
<b>it</b>	Compressione	Effetto proprioiettivo	Scarico
<b>es</b>	Compresión	Efecto propioceptivo	Descarga
<b>pt</b>	Compressão	Efeito proprioceptivo	Alívio do peso
<b>da</b>	Kompression	Proprioceptisk effekt	Aflastning
<b>fi</b>	Kompressio	Liikeaistiin kohdistuva vaikutus	Vapautus
<b>sv</b>	Kompression	Proprioceptiv effekt	Avlastning
<b>el</b>	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας	Αποφόρτιση
<b>cs</b>	Komprese	Proprioceptivní účinek	Snížení zátěže
<b>pl</b>	Kompresja	Propriocepcja	Odciążenie
<b>lv</b>	Kompresija	Proprioceptīvs efekts	Atslogošana
<b>lt</b>	Kompresija	Propriocepčinis poveikis	Apkrovos sumažinimas
<b>et</b>	Kompressioon	Proprioceptiivne toime	Koormuse vähendamine
<b>sl</b>	Kompresija	Proprioceptivni učinek	Razbremenitev
<b>sk</b>	Kompresia	Proprioceptívny účinok	Odfahčenie
<b>hu</b>	Kompresszió	Proprioceptív hatás	Tehermentesítés
<b>bg</b>	Компресия	Проприоцептивен ефект	Отбремняване
<b>ro</b>	Compressie	Efect proprioceptiv	Descărcare
<b>ru</b>	Компрессия	Проприоцептивный эффект	Разгрузка
<b>hr</b>	Kompresija	Učinak proprioceptivnog vježbanja	Rasterećenje
<b>zh</b>	压紧	本体感	减压
<b>ar</b>	الضغط	مفعول استقبال الحس العميق	تخفيف ثقل

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---

en

## **PROPRIOCEPTIVE ELASTIC ANKLE BRACE FOR ACHILLES TENDON**

### **Description/Destination**

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition**

Polyamide - elastane - polyester - polyurethane - thermoplastic elastomer - ethylene vinyl acetate - presence of natural latex.

### **Properties/Mode of action**

Elastic compressive knitting (class 3 - 20-36 mmHg / 271-48 hPa).

Insert at the Achilles tendon to protect the tendon and absorb shocks.

#### 2 stackable pairs of heelpieces:

- to offload the Achilles tendon,
- to balance the other leg,
- to allow 3 different offloading heights depending on the pathology.

Tabs to make it easier to pull on.

### **Indications**

Conservative treatment of Achilles tendon rupture.

Postoperative pain and edema of the Achilles tendon.

Prevention and treatment of Achilles tendon injuries: tendonitis, tenosynovitis, bursitis, achillodynia and Haglund's disease.

Prevention of sports injuries of the Achilles tendon.

### **Contraindications**

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

## **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

## **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## **Instructions for use/Application**

Two pairs of heelpieces are supplied with the ankle brace.

Make sure you use the same number of heelpieces (1 or 2) on both sides, following the advice of the physician who prescribed the device.

Slip the ankle brace over the foot, positioning the insert over the Achilles tendon.

If necessary, superimpose the two heelpieces, placing heelpiece ® (the slimmest one) on top of heelpiece   (the highest one).

Insert the base heelpiece ( ) or the two heelpieces (  and ®) into the shoe of the foot wearing the ankle brace. Repeat the same operation for the other shoe.

## **Care**

The product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Machine washable at 30°C (delicate programme). The device has been tested over 20 wash cycles. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

## **Storage**

Store at room temperature, preferably in the original packaging.

## **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2023503 (2025-03)
**THUASNE UK Ltd**
Thuasne SAS - SIREN/RCS Nanterre 542 091 186