THUASNE®

de Gebrauchsanweisung  
en Instructions for use  
fr Notice d'utilisation  
it Istruzioni per l'uso  
es Instrucciones de utilización  
pt Instruções de uso  
nl Gebruiksaanwijzing  
da Brugsanvisning  
sv Bruksanvisning  
fi Käyttöohje  
no Bruksanvisning  
el Οδηγίες χρήσης  
cs Návod k použití  
pl Sposób użycia  
et Kasutusjuhend  
lv Lietošanas instrukcija  
lt Naudojimo instrukcija  
sk Návod na použitie  
sl Navodila za uporabo  
hu Használati utasítás  
ro Instrucțiuni de utilizare  
bg Инструкции за употреба  
hr Uputa za uporabu  
mk Упатство за користење  
sr Uputstvo za korišćenje  
tr Kullanım Talimatları  
ru Инструкция по применению  
ko 사용 설명서  
ja 使用方法  
he הוראות שימוש  
ar دليل الاستخدام حزام  
zh 使用说明

SERENA S

fr	_____	3
en	_____	4
de	_____	5
it	_____	6
es	_____	7
pt	_____	8
nl	_____	9
da	_____	10
sv	_____	11
fi	_____	12
no	_____	13
el	_____	14
cs	_____	15
pl	_____	16
et	_____	17
lv	_____	18

lt	_____	19
sk	_____	20
sl	_____	21
hu	_____	22
ro	_____	23
bg	_____	24
hr	_____	25
mk	_____	26
sr	_____	26
tr	_____	27
ru	_____	27
ko	_____	28
ja	_____	28
he	_____	29
ar	_____	29
zh	_____	30

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**How to look after Serena 9201, 9341, 9301, 9311**

You have selected a quality product in choosing a Serena breast form. The soft, skin-friendly material has excellent properties when worn: consistency, elasticity and movement are very similar to the natural properties of the breast.

**Indication and intended purpose**

After breast surgery (mastectomy, breast-conserving surgery, breast reconstruction) and after full healing of the surgical wound and completed radiation therapy, Serena breast prostheses will give you a natural, cosmetically and visually harmonious appearance as well as a corresponding weight compensation.

Design and size depend on anatomical conditions. Consistency, elasticity and behaviour in motion are very similar to the characteristics of the natural breast. All materials used have been tested for skin tolerance.

**Contraindications**

None known

**Instructions for Use**

All Serena breast forms should be worn in a suitable bra, either pocketed or directly against the skin.

**Instructions for Care**

Daily care: Clean the Serena breast form in luke warm water using a mild detergent or soap. Never use a brush! To dry, wrap the Serena breast form in a hand towel and pat it lightly till dry. Do not place the form on a heater and never use a hair-dryer!

**Important Points**

Keep pointed and sharp objects away from the breast form. These can easily damage the delicate outer skin and make the form unserviceable. A repair is not possible. Avoid dropping the form onto a hard surface. Please always store your Serena in its original packaging!

Serena breast forms are resistant to chlorine and salt water. In rare cases, the consistency of the silicone may be affected by aggressive perspiration and bacteria. A slight loss in shape and discolouration may occur with time.

**Precautions**

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

**Product Warranty**

This Serena silicone product carries a 2-year warranty against manufacturing faults. Please make a note of the date of receipt to validate any future claim.

In the unlikely event of any fault occurring please return the breast form in its original packaging to your hospital or other supplier who will process the claim on your behalf.

Please note that this warranty does not cover products that have been incorrectly stored or exposed to damaging external influences, e.g., sharp objects, pet's claws, etc., or extend to adhesive pads and strips. We comply with all statutory requirements and regulations.

**Technical data for all silicone prostheses**

Polyurethane film, material on the basis of polydimethylsiloxane, colour pigments

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- CHN** Thuasne (Shanghai) Medical Devices Co., Ltd. · 159 Tianzhou Lu · Shanghai, 200233
Thuasne Deutschland GmbH · Sydney Garden 9 · D-30539 Hannover
#### Thuasne UK Ltd
### Thuasne Deutschland GmbH