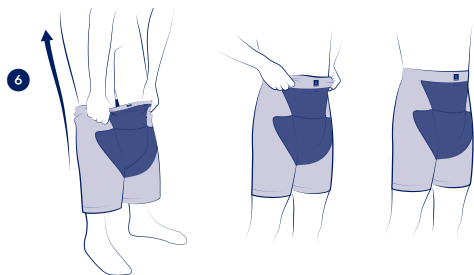
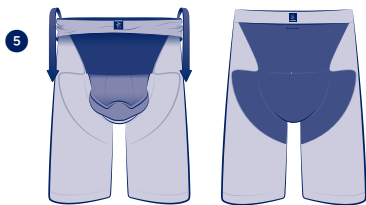
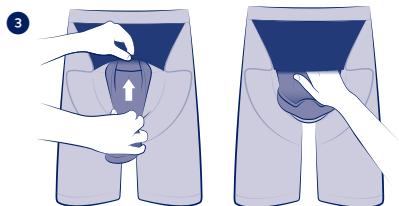
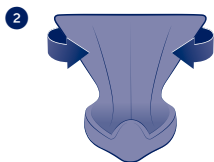
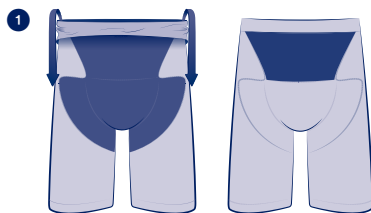
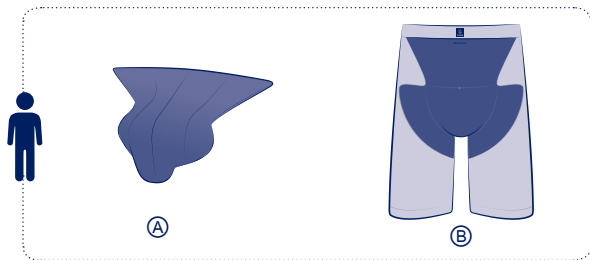
THUASNE®

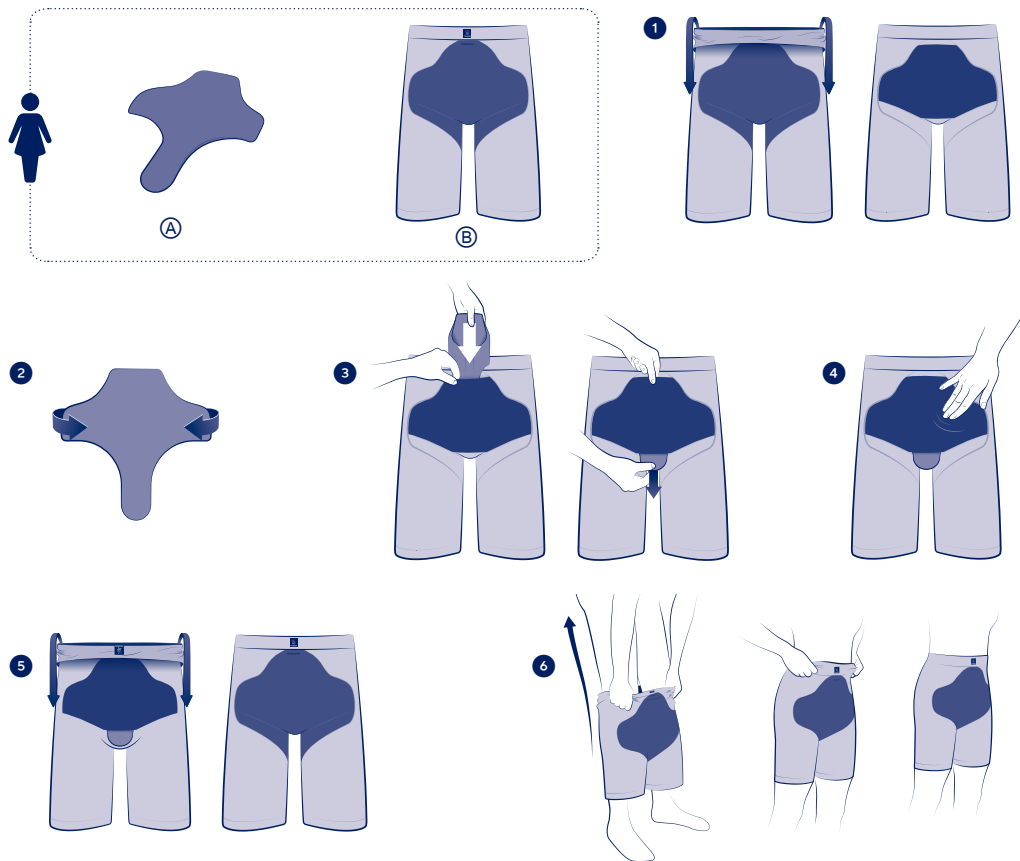
fr	Short de compression et de mobilisation.....	6
en	Mobilizing and compression shorts.....	7
de	Kompressions- und Mobilisierungs-Shorts.....	8
nl	Compressie- en mobilisatieshort.....	10
it	Dispositivo di compressione e di mobilitazione.....	11
es	Short de compresión y movilización.....	13
pt	Calções de compressão e de mobilização.....	14
da	Kompressions- og mobiliseringsshorts.....	15
fi	Kompressio- ja mobilisointishortsit.....	17
no	Kompresjonssshorts.....	18
sv	Shorts som stimulerar och aktiverar.....	19
el	Σορτς συμπίεσης και κινητοποίησης.....	21
cs	Kompresní a mobilizační šortky.....	22
pl	Spodenki uciskowe i mobilizujące tkanki.....	23
lt	Kompresiniai palaikomieji šortai.....	25
et	Surve- ja mobiliseerimisseeade.....	26
sl	Kompresijske in mobilizacijske kratke hlače.....	28
sk	Kompresívna a mobilizačná pomôcka.....	29
hu	Kompressziós és mobilizáló short.....	30
bg	Шорти за компресия и мобилизация.....	32
ro	Colanți scurți de compresie și de mobilizare.....	33
ru	Мобилизующие компрессионные шорты.....	34
hr	Kratke hlače za kompresiju i imobilizaciju.....	36
zh	具有压力和活血功效的短裤.....	37
ja	圧迫・弾性バンティ.....	38
ko	압박 및 모빌라이징 쇼트 팬츠.....	40
ar	شورت داخلي للضغط والتحرك.....	43

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Bruksanvisning - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - 使用方法 - 사용설명서 - إرشادات الاستعمال

		cK		cG	
		cm	in	cm	in
1		88 - 94	34.5 - 37	45 - 54	17.75 - 21.25
2		95 - 101	37.5 - 39.75	48 - 58	19 - 22.75
3		102 - 108	40.25 - 42.5	51 - 62	20 - 24.5
4		109 - 115	43 - 45.25	54 - 66	21.25 - 26
5		116 - 122	45.75 - 48	57 - 69	22.5 - 27.25
6		123 - 129	48.5 - 50.75	60 - 72	23.5 - 28.25

cK		cG
fr	Circonférence bassin	Circonférence haut de cuisse
en	Pelvis circumference	Upper thigh circumference
de	Beckenumfang	Umfang des oberen Oberschenkels
nl	Bekkenomtrek	Dijomtrek bovenaan
it	Circonferenza bacino	Circonferenza coscia nel punto più alto
es	Circunferencia de la pelvis	Circunferencia de la parte superior del muslo
pt	Circunferência da bacia	Circunferência da parte superior da coxa
da	Omkreds af bækken	Omkreds af den øverste del af låret
fi	Lantionympärys	Reidenympärys
no	Bekkenomkrets	Omkrets øverst på låret
sv	Omkrets runt höften	Omkrets runt lår
el	Περίμετρος λεκάνης	Περίμετρος άνω τμήματος μπουού
cs	Obvod boků	Obvod horní části stehna
pl	Obwód miednicy	Obwód górnej części uda
lt	Klubų apimtis	Šlaunies viršutinės dalies apimtis
et	Vaagna ümbermõõt	Reie ümbermõõt
sl	Obseg medenice	Obseg zgornjega dela stegen
sk	Obvod panvy	Obvod vrchnej časti stehna
hu	Csípőbőség	Felső combbőség
bg	Обиколка на таза	Горна обиколка на бедрото
ro	Circonfereință bazin	Circonfereință coapse
ru	Обхват таза	Обхват верхней части бедра
hr	Opseg zdjelice	Opseg gornjeg dijela bedara
zh	臀围	大腿围
ja	骨盤周囲	大腿上部周囲
ko	골반 둘레	허벅지 상단 둘레
ar	محيط الخوض	محيط أعلى الفخذ

fr

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Conserver cette notice.

en

MOBILIZING AND COMPRESSION SHORTS

Description

The Mobiderm® Intimate range offers a variety of mobilizing and compression garments for subcutaneous or indurated edema.

These garments are underwear designed to treat pelvic and/or genital edema. It is composed of:

- compression shorts ①,
- a removable Mobiderm® genital mobilization pad in small blocks ②.

Mobiderm® Intimate can be worn with or without the genital mobilization pad, depending on the patient's needs and the recommendations of the healthcare professional.

This range consists of a men's model and a women's model, each available in 6 sizes.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components: polyamide - polyester - elastane - acrylic adhesive mass.

Foam components: polyethylene (small blocks).

Properties/Mode of action

Mobiderm® Intimate mobilizing and compression garments provide:

- compression around the pelvis and buttocks, to encourage lymphatic drainage and help maintain or reduce swelling;
- subcutaneous drainage of edematous and indurated areas, by creating a pressure gradient between the parts of the skin compressed by the foam blocks of the Mobiderm® genital pad and the surrounding areas, which helps drain away the excess lymph fluid and reduce swelling.

The garments are designed to be worn day and night.

They provide graduated pressure (from the waist to the hips and thighs):

- 15-20 mmHg around the pelvis,
- 5-10 mmHg on the abdomen and mid thighs.

Indications

Treatment of pelvic and genital lymphedema.

Contraindications

Do not use in case of pelvic or of the leg major dermatological conditions.

Do not use in the event of known allergy to any of the components.

Do not use in the event of septic thrombosis.

Do not use in case of severe peripheral neuropathy of the affected limb.

Do not use in the event of phlegmasia cerulea dolens (painful blue phlebitis with arterial compression) of an affected limb.

Do not use in case of lower limb peripheral arterial disease (PAD) with ankle-brachial index (ABI) < 0.6.

Do not use for the patients with significant edema localized only in the labia minora.

Do not use in case of posthectomy less than 3 months old.

Do not use for pregnant women (from the first month).

Do not use in the event of decompensated heart failure.

Do not use in case of extra-anatomical bypass of the affected limb.

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

The risk/benefit ratio and the adequate level of pressure must be regularly re-assessed by a healthcare professional in the event of:

- PAD with ABPI between 0.6 and 0.9
- Advanced peripheral neuropathy
- Weeping or eczematous dermatitis.

In the event of discomfort, significant restriction, pain, variation in limb volume, deterioration of skin condition, infection, unusual sensations, change in the colour of the extremities, or changes in the performance of the product, remove the product and seek the advice of a healthcare professional.

Do not apply the product to an open wound.

Due to the constant evolution of the pathology, regular follow-up with a healthcare professional is recommended to check that the compression device is still suitable for the pathology.

Do not use products on the skin (creams, ointments, oils, gels, patches...) before applying the device, it could damage the product.

Take care not to damage the device when fitting it, particularly with your nails. In the event of associated advanced venous disease or venous ulcer or lymphedema, seek the advice of a healthcare professional for appropriate compression treatment on the entire leg.

When worn with compression stockings, it is recommended to place the device over the stocking.

The genital mobilization pad is designed exclusively for use with Mobiderm® Intimate. The two items should be used together.

Do not use when swimming.

Do not let the garment contact the vaginal mucosa (e.g. edema of the labia minora).

For hygiene, security and performance reasons, do not re-use the product for another patient.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity or cutaneous dryness.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

- We recommend washing your new garment before use.
- Verify the product's integrity before every use.
- Mobiderm® Intimate should be worn against the skin, like an undergarment.
- The genital mobilization pads are reversible.
- Choose the appropriate size to fit the patient, referring to the size chart.

Inserting the male genital mobilization pad:

- The genital mobilization pad should be inserted before putting on the shorts.
- Turn the shorts inside out and locate the inner pocket over the groin. ❶
- Place the genital pad face up with the concave section towards the bottom, and fold the corners into the middle. ❷
- Insert the genital pad into the pocket, placing the top in first then sliding the pad upwards. ❸
- Unfold and adjust the genital pad, making sure it fits neatly into the pocket. ❹
- Turn the shorts right-side out. ❺

Inserting the female genital mobilization pad:

- The genital mobilization pad should be inserted before putting on the shorts.
- Turn the shorts inside out and locate the inner pocket over the groin. ❶
- Place the genital pad flat with the tab towards the bottom, and fold the wings into the middle. ❷
- Insert the genital pad into the pocket at the waist, placing the narrowest part in first then sliding the pad down until the tab is sticking out of the other side of the pocket. ❸
- Unfold and adjust the genital pad, making sure it fits neatly into the pocket. ❹
- Turn the shorts right-side out. ❺

Putting on the shorts:

- Place both legs in the shorts, then pull the garment gently and evenly up towards the waist. ❶
- Men: when pulling up the panty, we recommend holding the genital pad by the tips of the triangle in order to keep it in place.
- Avoid tugging on the fabric or waistband more than necessary.
- The pocket containing the genital pad should sit at the front.
- Adjust the garment to make sure there are no folds, do not roll up the bottom of the garment.

Care/Maintenance

To ensure your compression garment lasts as long as possible and to prevent infection, it is important to wash it every day, or after each use. Mobiderm® Intimate mobilizing and compression garments are designed for daily use over six months, or for use on alternate days over a year i.e. 180 washes. After this time, garments should be replaced to ensure the treatment is effective and technical performance is not affected.

Machine washing:

- Wash the shorts and genital pad on a normal cycle at 40°C.
- Use a laundry detergent without fabric softener.
- Do not use chlorine-based detergents or harsh products.
- Remove the genital mobilization pad from the shorts.
- Place the shorts and genital pad in a mesh laundry bag to prevent damage by other garments (e.g. snagging on touch-fastening tape or zippers).
- Air dry, away from direct heat (e.g. radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2050002 (2025-05)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186