THUASNE®

Epi-GO

de	Ellenbogenstütze gegen Epicondylitis	5
en	Anti-epicondylitis elbow brace	6
fr	Coudière anti-épicondylite	7
nl	Anti-epicondylitis elleboogbrace	8
it	Gomitiera antiepicondilite	9
es	Codera antiepicondilitis	10
pt	Cotoveleira antiepicondilite	11
da	Albueunderlag mod tennisalbue	13
fi	Kyynärtuki epikondyliitin hoitoon	14
sv	Armbåggsskydd mot epikondylit	15
el	Καλύπτρα αγκώνα κατά της επικονδυλίτιδας	16
cs	Loketní ortéza proti epikondylitidě	17
pl	Orteza stawu łokciowego przeciwko entezopatii nadkłykcia	18
lv	Antiepi kondilīta elkoņa pārsējs	19
lt	Alkūnės įtvaras epikondilitui gydyti	21
et	Epikondüliidi küünarnukitugi	22
sl	Komolčna blazinica proti epikondilitisu	23
sk	Antiepi kondylická bandáž	24
hu	Teniszkönyök elleni könyökrögzítő	25
bg	Налакътник срещу епикондилит	26
ro	Cotieră anti-epicondilită	27
ru	Налокотник против воспаления надмыщелка	29
hr	Anti-epikondilitis štitnik za laktove	30
zh	抗上髁炎护肘	31
ar	واقى مرفق مضاد لالتهاب اللقيمة	33

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال


	0	21 - 23 cm
	1	23 - 25 cm
	2	25 - 27 cm
	3	27 - 29 cm
	4	29 - 31 cm
	5	31 - 34 cm


de	Unterarmumfang (5 cm unterhalb des Ellenbogens)
en	Circumference 5 cm under the elbow
fr	Circonférence 5 cm sous le coude
nl	Omtrek 5 cm onder de elleboog
it	Circonferenza 5 cm sotto il gomito
es	Circunferencia de 5 cm bajo el codo
pt	Circunferência 5 cm abaixo do cotovelo
da	Omkreds 5 cm under albuen
fi	Ympärysmitta 5 cm kyynärpään alapuolelta
sv	Omkrets 5 cm under armbågen
el	Περίμετρος 5 εκ. κάτω από τον αγκώνα
cs	Obvod 5 cm pod loktem
pl	Obwód 5 cm poniżej łokcia
lv	Apkārtmērs 5 cm zem elkoņa
lt	Apimtis 5 cm žemiau alkūnės
et	Küünarnuki all ümbermõõt 5 cm
sl	Obseg 5 cm pod komolcem
sk	Obvod 5 cm pod lakťom
hu	Kerület könyök alatt 5 cm-rel
bg	Обиколка 5 cm под лакътя
ro	Circumferință de 5 cm sub cot
ru	Окружность 5 см ниже локтя
hr	Opseg 5 cm ispod lakta
zh	肘部下方周长5 厘米
ar	محيط بمقدار 5 سم أسفل المرفق

de	Entlastung
en	Support
fr	Maintien
nl	Fixatie
it	Sostegno
es	Sujeción
pt	Manutenção
da	Støtte
fi	Tuki
sv	Stöd
el	Συγκράτηση
cs	Stabilizace
pl	Podtrzymywanie
lv	Atbalsts
lt	Palaikymas
et	Hooldus
sl	Oprijem
sk	Podpora
hu	Támasz
bg	Поддръжка
ro	Suținere
ru	Стабилизация
hr	Potpora
zh	稳定
ar	الدعم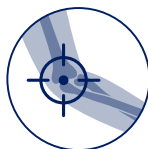


de	Kompression	Propriozeptive Wirkung	Gezielter Druck
en	Compression	Proprioceptive effect	Targeted pressure
fr	Compression	Effet proprioceptif	Pression localisée
nl	Compressie	Proprioceptieve werking	Lokale druk
it	Compressione	Effetto propriocettivo	Pressione localizzata
es	Compresión	Efecto propioceptivo	Presión localizada
pt	Compressão	Efeito propioceptivo	Pressão localizada
da	Kompression	Proprioceptisk effekt	Lokalt tryk
fi	Kompressio	Liikeaistiin kohdistuva vaikutus	Paikallinen paine
sv	Kompression	Proprioceptiv effekt	Lokaliserat tryck
el	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας	Εντοπισμένη πίεση
cs	Komprese	Proprioceptivní účinek	Lokalizovaný tlak
pl	Kompresja	Propriocepcja	Ucisk miejscowy
lv	Kompresija	Proprioceptīvs efekts	Lokalizēts spiediens
lt	Kompresija	Propriocepcinis poveikis	Konkrečios vietos suspaudimas
et	Kompressioon	Proprioceptiivne toime	Paikne surve
sl	Kompresija	Proprioceptivni učinek	Lokaliziran pritisk
sk	Kompresia	Proprioceptívny účinok	Presne zacielený tlak
hu	Szorítás	Proprioceptív hatás	Lokalizált nyomás
bg	Компресия	Проприоцептивен ефект	Локализирано налягане
ro	Compresie	Efect proprioceptiv	Presiune localizată
ru	Компрессия	Проприоцептивный эффект	Локализованное давление
hr	Kompresija	Učinak proprioceptivnog vježbanja	Lokalizirani pritisak
zh	压紧	本体感	局部施压
ar	الضغط	مفعول استقبال الحس العميق	ضغط موضعي

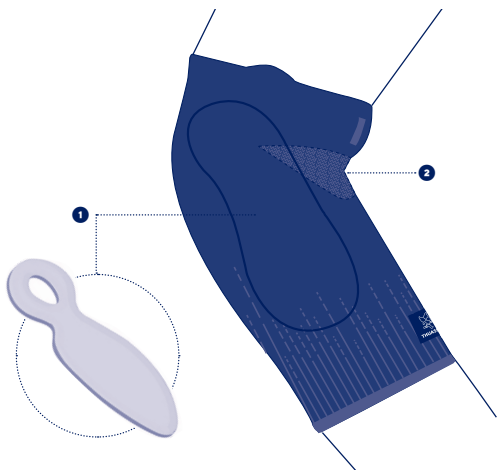
A

B

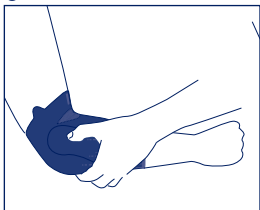C

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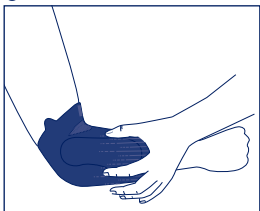ANTI-EPICONDYLITIS ELBOW BRACE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

This device is an elastic proprioceptive elbow brace providing targeted pressure to relieve tendon pain.

This device can be used as a proprioceptive support for rehabilitation and particularly when resuming sports activities.

Composition

Textile components: polyamide (81%) - elastane (13%) - polyester (6%)

Non-textile components: polyurethane - polyethylene foam

Properties/Mode of action

Elbow support with anatomically-shaped inserts. ❶

Anatomically-shaped knit for a good fit. ❷

Indications

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Slip on the elbow support, positioning the soft blue knitted zone in the crease of the elbow, with the pull tabs upwards. ①

Position the inserts on either side of the elbow and centre the holes over the epicondyles (medial and lateral). ② ③

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label.

Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Machine washable at 30 °C (delicate programme). Do not tumble-dry. Do not iron. Do not dry clean. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Τοποθετήστε τα παρεμβλήματα από την μια και την άλλη πλευρά του αγκώνα και ευθυγραμμίστε τα κενά στο επίπεδο των κονδύλων (μεσαίου και πλευρικού). Ⓑ ©
THUASNE DEUTSCHLAND GmbH
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186