THUASNE®

Malleo-GO

de	Elastische propriozeptive Knöchelbandage mit Knöchelschutz	5
en	Proprioceptive elastic ankle brace with malleoli protection	6
fr	Chevillère élastique proprioceptive avec protection des malléoles	7
nl	Proprioceptieve elastische enkelbrace met malleoli bescherming	8
it	Cavigliera elastica propriocectiva con protezione dei malleoli	10
es	Tobillera elástica propioceptiva con protección de los maléolos	11
pt	Tornozeleira elástica propriocectiva com proteção dos maléolos	12
da	Proprioceptisk, elastisk ankelbind med beskyttelse af malleolerne .	13
fi	Proprioseptiivinen joustava nilkkatuki, jossa kehräsluiden suoja	14
sv	Proprioceptiv elastisk ankelortos med skydd för fotknölarne	16
el	Ελαστική ιδιοδεκτική επιστραγαλίδα με προστασία των σφυρών ...	17
cs	Elastická proprioceptivní ortéza kotníku se zesílenou ochranou v oblasti kotníku	18
pl	Elastyczna proprioceptywna orteza skokowo-stopowa z zabezpieczeniem kostek	19
lv	Proprioceptīvs elastīgs potītes pārsējs ar potīšu kaulu aizsardzību ..	20
lt	Tamprus propriocepriją gerinantis čiurnos įtvaras su čiurnakaulių apsaugos zonomis	22
et	Proprioceptiivne elastne pahkluside malleoolikaitsetega	23
sl	Elastična proprioceptivna opornica z zaščito za gleženj	24
sk	Elastická proprioceptívna členková bandáž s ochranou členkov	25
hu	Proprioceptív rugalmas bokarögzítő bokavédővel	26
bg	Проприоцептивна еластична шина за глезен със защита на малеолите	28
ro	Glezniară elastică proprioceptivă cu protecție pentru maleole	29
ru	Эластичный проприоцептивный ортез с защитой лодыжки	30
hr	Elastični proprioceptivni steznik za gležanj s maleolarnom zaštitom	32
zh	帶有踝骨保护的彈力本体感护踝	33
ar	دعامة كاحل مرنة لاستقبال الحس العميق مزودة بحماية للوعيين	35

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	0	16 - 19 cm
	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

de	Knöchelumfang
en	Ankle circumference
fr	Tour de cheville
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
ru	Окружность голени
hr	Opseg gležnja
zh	脚踝尺寸
ar	محيط الكاحل

de	Entlastung
en	Support
fr	Maintien
nl	Fixatie
it	Sostegno
es	Sujeción
pt	Manutenção
da	Støtte
fi	Tuki
sv	Stöd
el	Συγκράτηση
cs	Stabilizace
pl	Podtrzymywanie
lv	Atbalsts
lt	Palaikymas
et	Hooldus
sl	Oprijem
sk	Podpora
hu	Támasz
bg	Поддръжка
ro	Suținere
ru	Стабилизация
hr	Potpورا
zh	穩定
ar	الدعم

de	Kompression
en	Compression
fr	Compression
nl	Compressie
it	Compressione
es	Compresión
pt	Compressão
da	Kompression
fi	Kompressio
sv	Kompression
el	Συμπίεση
cs	Komprese
pl	Kompresja
lv	Kompresija
lt	Kompresija
et	Kompressioon
sl	Kompresija
sk	Kompresia
hu	Szorítás
bg	Компресия
ro	Compresie
ru	Компрессия
hr	Kompresija
zh	压紧
ar	الضغط

Propriozeptive Wirkung
Proprioceptive effect
Effet proprioceptif
Proprioceptieve werking
Effetto propriocettivo
Efecto propioceptivo
Efeito propioceptivo
Proprioceptisk effekt
Liikeaistiin kohdistuva vaikutus
Proprioceptiv effekt
Αποτέλεσμα ιδιοδεκτικότητας
Proprioceptivní účinek
Propriocepcja
Proprioceptīvs efekts
Propriocepcinis poveikis
Proprietseptiivne toime
Proprioceptivni učinek
Proprioceptivny účinok
Proprioceptív hatás
Проприоцептивен ефект
Efect proprioceptiv
Проприоцептивный эффект
Učinak proprioceptivnog vježbanja
本体感
مفعول استقبال الحس العميق

de

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en

PROPRIOCEPTIVE ELASTIC ANKLE BRACE WITH MALLEOLI PROTECTION

Description/Use

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

This device can be used as a proprioceptive support for rehabilitation and particularly when resuming sports activities.

Composition

Textile components: polyamide (83%) - elastane (14%) - polyester (3%)

Non-textile components: polyurethane - polyethylene foam

Properties/Mode of action

Elastic compressive knitting (class 3 - 20-36 mmHg / 271-48 hPa).

Foam padding: protection of the malleoli.

Indications

Mild ankle sprain (acute phase).

Post-operative, post-traumatic residual pain and/or edema.

Joint instability/laxity.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Pull the ankle brace over the ankle joint, like a sock.

Place the malleoli correctly above the foam inserts.

The foam insert should not lie on the malleoli.

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Machine washable at 30°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Composition

Composants textiles : polyamide (83%) - élasthanne (14%) - polyester (3%)

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©Thuasne - CO0060000701 (2026-02)
THUASNE DEUTSCHLAND GmbH
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186