**THUASNE®**

# Osteomed Acute

de Gebrauchsanweisung - Osteoporose-Orthese  
en Instructions for use - Osteoporosis orthosis  
fr Consignes d'utilisation - Orthèse traitant l'ostéoporose  
it Istruzioni per l'uso - Ortesi per osteoporosi  
es Instrucciones de utilización - Ortesis contra la osteoporosis  
pt Instruções de uso - Órtese para osteoporose  
nl Gebruiksaanwijzing - Osteoporose-orthese  
da Brugsanvisning - Osteoporose-ortose  
sv Bruksanvisning - Osteoporos-ortos  
fi Käyttöohje - Osteoporositukside  
el Οδηγίες χρήσης - Πολυθεραπευτικός κηδεμόνας οστεοπόρωσης  
cs Návod k použití - Stabilizační ortéza  
pl Sposób użytkowania - Orteza stabilizacyjna  
et Kasutusjuhend - Stabiliseerimisortoos  
lv Lietošanas instrukcija - Stabilizējošā ortoza  
lt Naudojimo instrukcija - Stabilizuojantis įtvaras  
sk Návod na použitie - Stabilizačná ortéza  
sl Navodila za uporabo - Stabilizacijska ortoza  
hu Használati utasítás - Stabilizáló ortézis  
ro Instrucțiuni de utilizare - Orteză de stabilizare  
bg Инструкция за употреба - Стабилизираща ортеза  
hr Upute za uporabu - Stabilizacijska ortoza  
ru Инструкция по использованию - Корсет стабилизирующий  
пояснично-крестцовый

(1)

(2)

(3)

(4)

(5)

(6)

(7)

(8)

(9)

(10)

(11)

(12)

(15)

(13)

(16)

(14)

(17)

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en

## How it works

The Osteomed Acute osteoporosis orthosis is a spine straightening orthotic device; its functionality has been designed to reflect the latest osteoporosis guidelines issued by the German Association for Osteology (Dachverband für Osteologie, DVO). The orthosis helps stabilise both defective/degenerative vertebrae and the entire spine from the upper thoracic vertebrae down to the sacral area. The spine is actively straightened and pain is reduced. Movement is also restricted at the sagittal level.

## Purpose

Osteomed Acute system is a combination of a back splint and belt system that offers active relief and correction of the lumbar and thoracic spine at the sagittal level. The orthosis may only be used for back treatment.

## Indications

- Osteoporosis
- Vertebral osteoporotic fractures
- Humpbacks
- Palliative therapy

## Contraindications

- Instable vertebral osteoporotic fractures
- Large-scale skin injuries or diseases
- Extensive haematoma and severe swellings

## Side effects

To date, no side effects have been registered during proper use.

## Fitting instruction (Technical adviser/Medical equipment supplier) (Fig. 1 - 11)

Required body measurements

1. Body circumference (widest part of the body)
2. Height of the back (from C7 - bottom cleft) (1)

Recommendation: Deduct approx. 11 - 14 cm from the measured back length (position the splint approx. 4 cm below C7 and approx. 7 - 10 cm above the bottom cleft).

Fitting the orthosis

3. Adjust the body circumference

Lay the orthosis flat on a table (2) and unfasten the side sections of the body from the back, measure to the required total length and shorten the side parts as evenly as possible (3) before re-fastening them (4).

4. Back length

Adjust the back splint to the measured length (by opening and closing the screws). Align the splint with the back contour, positioning it always 4 cm below C7 and push back into the back pocket (5). Shorten the pocket accordingly (6) and close up the upper pocket fastening (7).

5. Putting on the orthosis, adjusting the length of the belts:

Put the orthosis on as you would a backpack. Close the front fastening of the lumbar bandage. For easier closing the patient can slide his or her hands into the loops of the front fastening (8).

Check the position of the back splint; it should rest accurately against the spine. The upper edge of the splint should not protrude above C7, the lower edge of the splint should end in the upper third of the sacrum (rule of thumb: 7 - 10 cm above the bottom cleft).

Check that the belt straps are not twisted.

The patient should now tension the belt straps using both hands by first pulling the belt straps evenly towards the front (9) and then fastening them to the front fastening (10). The patient should notice how the spine feels tangibly, but still pleasantly straighter. Adjust the length of the shoulder straps and shorten if necessary (11).

## Putting on the orthosis (patients) (Fig 12 - 17)

Wear the Osteomed Acute over clothing (shirt/blouse, sweater), not on your skin. You may find it helpful to stand in front of a mirror when putting on the orthosis.

The Osteomed Acute is put on using the same principle as a backpack:

- Open the large front fastening.
- Pull on the orthosis as you would a backpack.
- Put your left and right hands in the loops of the front fastening (12) and close the two halves of the fastening (13).

Now check in front of the mirror:

- Check the position of the back splint (14); it should rest accurately against your spine. The lower edge of the splint should be slightly above the bottom cleft.
- Check that the belt straps (15) are not twisted.
- Now tension the belt straps using both hands. To do this, first pull the belts evenly towards the front (16) and then fasten them to the front fastening (17).

## Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

## Care instructions

- Please always wash the orthosis separately and do not soak.
- Remove the back splint before washing and fasten the velcro systems.
- Clean the splint with a moist cloth
- "Reshape" the orthosis after washing and dry immediately.

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**Thuasne UK Ltd**
**Thuasne Deutschland GmbH**